

A unique approach to difficult-to-dress wounds

Triad is a sterile coating that can be applied anywhere, adheres to wet skin, and keeps the wound covered.



Wounds on **irregular surfaces**



Broken skin in the presence of incontinence



Maceration of periwound skin




Necrotic tissue such as slough or eschar


TRIAD APPLICATION:



1. Always cleanse the wound before applying Triad.

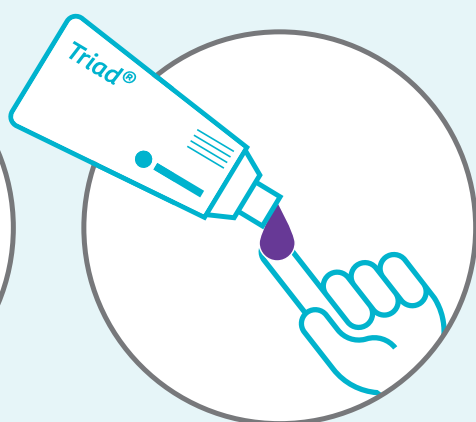

2. Triad can be applied directly from the tube or by using a gloved finger.

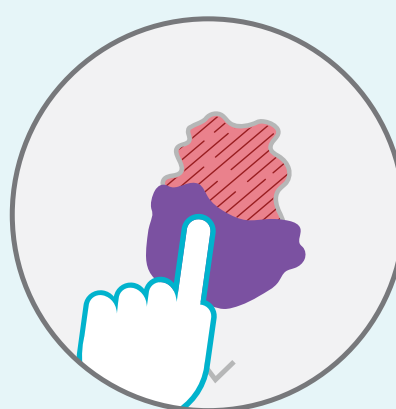

3. Gently spread Triad evenly over the area of application to the thickness of a dime.


Because Triad is designed for use on wet and irregular surfaces, it is **typically used on its own without a secondary dressing.**

Triad may not always **dry completely** and some light staining on bed linens or garments might occur.


Triad can stay in place for **up to 7 days**, with higher exudate levels requiring more frequent re-applications.

In the **perineal area**, reapply Triad after each episode of incontinence. Triad does not need to be removed completely before reapplying.


TRIAD REMOVAL:

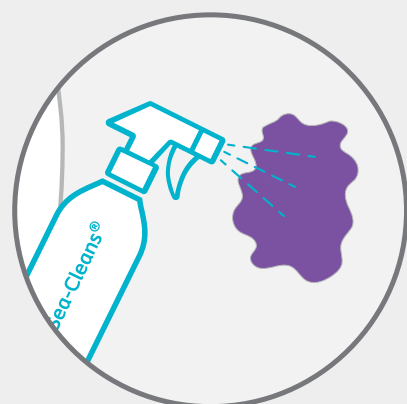

1. Use pH-balanced wound cleanser to soften Triad.

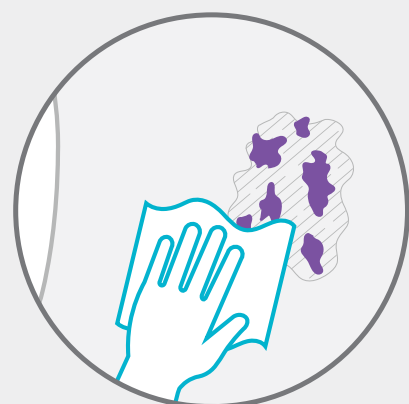

2. Gently wipe to remove without scrubbing.

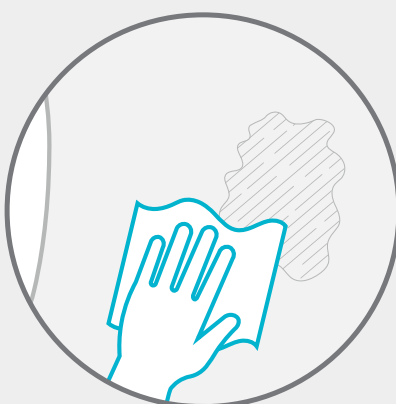


3. For complete removal repeat as needed.

Play the Triad Training Game

Try the game to see examples of wound types Triad is indicated for. It's quick and easy to learn how to apply and remove Triad!

Scan to get started.

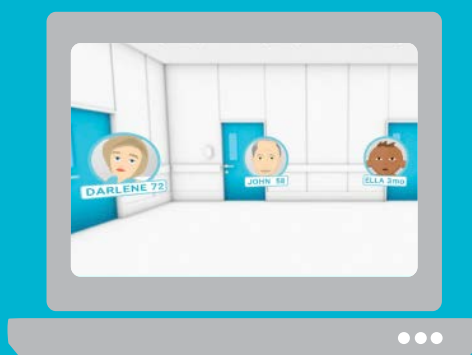

Triad instructions for use video

Scan to view the video.



Ordering Information

Triad® Hydrophilic Wound Dressing

| Code | Size | Units | HCPS Code |
|------|----------------|-------|-----------|
| 1964 | 2.5 oz. (71 g) | 12 | A6240 |
| 1967 | 6 oz. (170 g) | 12 | A6240 |



Sea-Cleans® Wound Cleanser

A pH balanced wound cleanser that can be used to remove Triad without scrubbing.

| Code | Size | Units | HCPS Code |
|------|---------------------|-------|-----------|
| 1063 | 6 fl. oz. (178 mL) | 12 | A6260 |
| 1067 | 12 fl. oz. (355 mL) | 12 | A6260 |



Triad is only sterile on first application. See complete product instructions for use.

Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast United States
Minneapolis, N 55411 / 1-800-533-0464
www.coloplast.us

Coloplast Canada
Oakville, ON L6H 5S9 / 1-877-820-7005
www.coloplast.ca

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2021 Coloplast Corp. All rights reserved.